

BAHA 22 listener – for Cochlear and Oticon Medical

This listening kit is suitable for use with bone anchored processors produced by Cochlear and Oticon Medical.

For daily testing by a carer/parent

Take the mount out of the carry case and position at least 150mm from case to avoid sound reflections.

Clip user's processor to the top of the mount as shown.



Crescendo settings: **Mic – Ext, Tone – set to Mid Point**

Check headphones are plugged into phones socket and the mount is plugged into mic socket.



Leave the processor as set for user and turn on Crescendo.

Listen through the headphones to check each day that the sound is consistent.



In normal use, the Crescendo should be able to remain in the foam cutout – all that is necessary is to twist the knob to 'OFF' when done.

If a significant change of sound is noticed, you should consult your audiologist.


IMPORTANT: Mounting procedure

Mounting

Hold the sound processor at an angle and carefully push it onto the mount.



DO NOT push the sound processor straight down onto the mount or **you may damage your sound processor.**

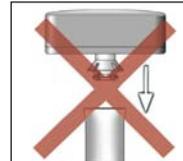


Releasing

Release the sound processor by tipping it slightly in any direction.



DO NOT pull the sound processor straight out or **you may damage your sound processor.**

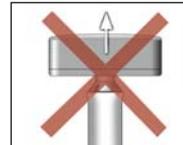

Cochlear processors fit inside the mount



Oticon Medical processors fit over the mount



Note: A mount with a green base is suitable for testing a BAHA from Cochlear or Oticon Medical. A mount with a blue base is only for testing a BAHA from Cochlear.



Connevens Limited

Bridge House, 1 Nutfield Road, Merstham, Surrey RH1 3EB
Information website: www.connevens.com

Helpline: **01737 247571**

Shop online: www.DeafEquipment.co.uk

Email: info@connevens.com