



**Connevens.info**

Solutions to improve the quality of life

*Offering you choice  
Helping you choose*

This product may be purchased from Connevens Limited  
secure online store at [www.DeafEquipment.co.uk](http://www.DeafEquipment.co.uk)

**DeafEquipment.co.uk**



## Technical specifications

**Neckloop diameter:** 22 cm, 9"

**Weight:** 46 g, 1.6 oz

### Cable length

BE9159: 90 cm, 3'

BE9161: 15 cm, 6"

### Connectors

3.5 mm tele plug (stereo) gold plated connector, 90 degrees angle  
(Break-away connector on cable)

**Load impedance:** 2 x 5  $\Omega$

### Magnetic output

1500 mA/m @ 15 cm, 6" distance and  
2 x 50 mW input signal

#### European office

Bellman & Symfon Europe AB  
Södra Långebergsgatan 30  
421 32 Västra Frölunda  
Sweden  
Phone +46 31 68 28 20  
Fax +46 31 68 28 90  
E-mail info@bellman.com

#### Asia Pacific office

Bellman & Symfon Asia Ltd.  
1401-06, Main Tower, GD Int'l Hotel  
339 Huanshi Dong Rd  
Guangzhou, China, 510095  
Phone +86 20 8318 2950  
Fax +86 20 8333 5528  
E-mail info.cn@bellman.com


© 2016, Bellman & Symfon AB, all rights reserved. Bellman, Bellman & Symfon and the Bellman & Symfon logo are registered trademarks of Bellman & Symfon AB.

## DESIGN FOR EARS™



[bellman.com](http://bellman.com)

BE9159\_006MAN008

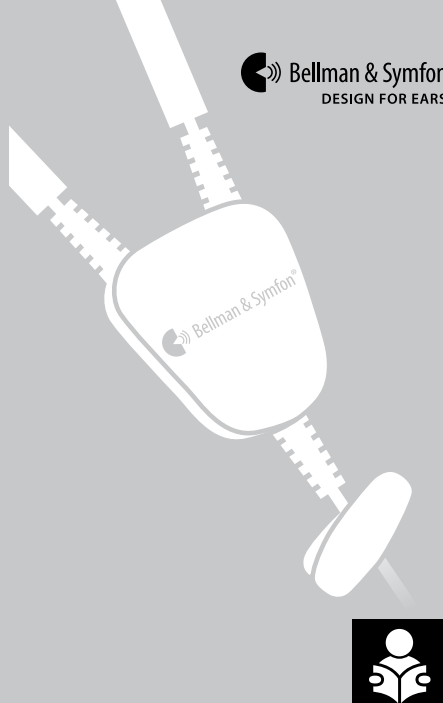


## User information

- This Neckloop from Bellman & Symfon is to be used with hearing aids equipped with Telecoil.
- It is optimized for the Maxi, Sonido, Mino speech amplifiers and the Domino listening system.
- The neckloop can also be used with smart devices, TVs, radios or other equipment that features a 3.5 mm stereo headphone jack intended for low impedance headphones.
- It is discreet and comfortable, and can easily be worn under your shirt. It also incorporates a breakaway neck strap for increased safety.
- **Important!** If you have a pacemaker, we recommend you check with your general practitioner or cardiologist before using a neckloop.



# Neckloop

BE9159/BE9161

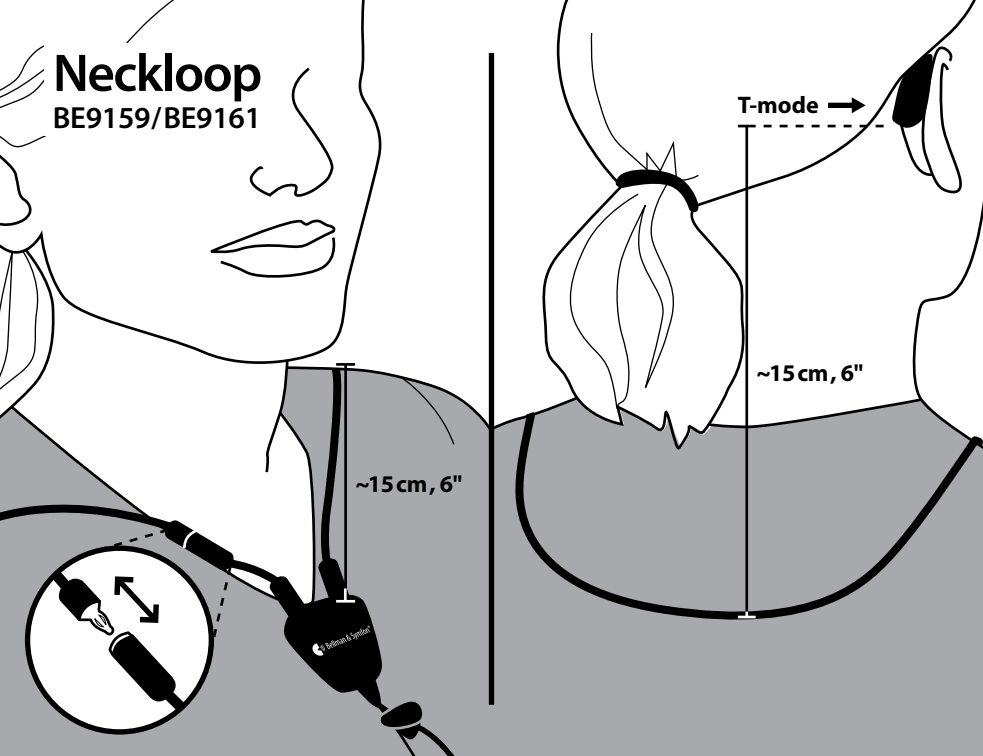

## Use with Domino or Mino

