


Tutor courtesy card - CRM-220 direct input



The CRM-220 radio system can be a great help to a hearing aid user - a little of your time to help understand what does what will be much appreciated.

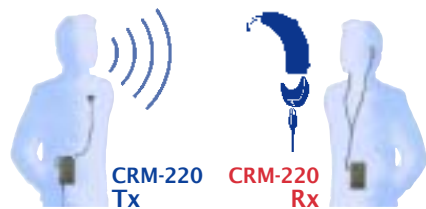

The tutor wears the radio microphone transmitter (Tx) to pick up their voice, the student wears a radio receiver (Rx) to feed sound directly into their hearing aid. The result is a greatly improved and relaxed listening experience for the hearing aid user.

There is no need to be self conscious in wearing a radio microphone - all the best TV stars do so.

It is important that you are comfortable wearing the CRM-220. Please ask for a different harness if you wish.



6 information points

- 1 The switch should be in the ON position. The battery low indicator should not be lit.



- 2 Microphone should be positioned about the length of a biro from the mouth.



- 3 Avoid wearing 'noisy' clinking jewellery.



- 4 During the lesson the CRM-220 will *automatically* adjust to louder voice levels so you need not change the way you normally talk.



- 5 During the lesson try not to draw attention to the CRM-220 user but do help them by always repeating questions from the floor.

- 6 Be aware that you have a direct sound link to CRM-220 users in the room and either turn off the transmitter or mute the microphone when you are talking to others - see following page for further details.

Tutor courtesy card *continued*

CRM-220

Muting the transmitter microphone

There are two methods:

- a) Turn off the CRM-T220 using the switch on the top panel. When turned on again the low battery light should give a confidence wink to show that the battery has adequate charge left.
- b) For some hearing aid users, physically turning off the CRM-220 transmitter can result in noises being heard from the receiver. Such noises are more prone to occur when used with digital hearing aids or cochlear implants. To avoid these noises, a separate switch to mute the microphone without turning the whole unit off is available. Press the switch once to mute the mic and once more to return the mic live.



From summer 2005: special direct input leads for the CRM-220 are available with inline RF interference filters to reduce the occurrence of 'noises' from digital hearing aids.

If the low battery indicator lights, the battery needs replacing.



Connection facilities into AV equipment are available, please discuss your requirements with your audiology support contact.



After the lesson, if you are not returning the transmitter to the user, remember to turn it off before going to the staff room or toilet!

The CRM-220 has a transmitting range of several classrooms.



Thank you for your time.

PS - general interest suggestion: ask your audiology contact to demonstrate what a hearing aid sounds like and how the CRM-220 system helps.

B220CCDI V1.0



CONNEVANS LIMITED

54 Albert Road North, Reigate, Surrey RH2 9YR
Helpline 01737 247571

Information: www.connevens.com

Online shopping: www.DeafEquipment.co.uk

info@connevens.com