

# CRM-220 radio system and neck loop for visiting hearing aid users



**This loan equipment will enable you to hear more clearly and help you to enjoy your visit.**




The speaker wears the radio microphone transmitter to pick up their voice



You wear the radio receiver to feed the sound directly to your hearing aid via the neck loop

## The receiver and its controls

- Set the switch of the receiver to **CHANNEL** as shown.
- Set your hearing aids to the "T" setting.
- Adjust the **volume** (marked Tx) of the receiver until you can hear clearly at a comfortable level through your hearing aids.

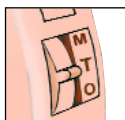

On/off & channel selection switch

Neck loop plugs in here

Volume control



**Congratulations – you should now be able to relax and just enjoy listening!**

## INFORMATION POINTS


Battery low indicator



- 1 The switch on your receiver should be pointing to the channel setting. The red battery low indicator should NOT be lit. If it lights up – ask for a new battery.



- 2 The speaker wears the radio microphone transmitter on a neck harness so that the microphone of the transmitter is positioned about the length of the biro from the mouth. If it is too far from the speaker's mouth, you will not get such a clear signal.

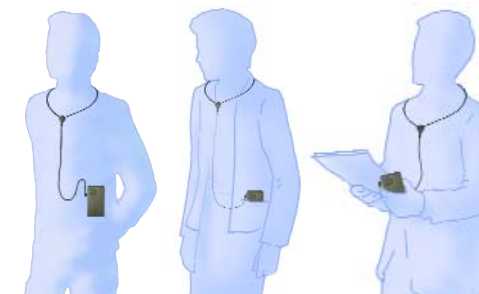

- 3 The radio receiver can either be clipped to your belt, placed safely in a pocket or held in your hand.

- 4 Once you have adjusted the Tx volume setting of the receiver to suit you personally, the CRM220 will automatically adjust to louder (or quieter) voice levels so you should not need to readjust the volume of the receiver or your hearing aids. Your hearing aids should remain on the "T" setting as long as you wish to listen to the speaker wearing the transmitter.

5. In some areas, the neck loop system may suffer interference from other electrical equipment such as lighting, computers, etc. If you should experience such interference, simply move to a position further away from the source of interference.

## Connevens have two websites that you may find of interest



[www.connevens.com](http://www.connevens.com) – this website has lots of information plus our entire catalogue online

[www.DeafEquipment.co.uk](http://www.DeafEquipment.co.uk) – the shopping website for all Connevens products plus online special offers