

Tutor courtesy card – Campus SX/MicroMLxS


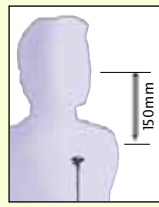

The Campus SX/MLxS radio system can be a great help to a hearing aid user – a little of your time to help understand what does what will be much appreciated.




The tutor wears the Campus SX radio microphone to pick up their voice, the student wears an MLxS radio receiver to feed sound directly into their hearing aid. The result is a greatly improved and relaxed listening experience for the hearing aid user.

There is no need to be self conscious in wearing a radio microphone – all the best TV stars do so.

6 information points

- 1 The student should give you the Campus SS transmitter turned on and ready to use – if you can see the display then the unit is on. 
- 2 Microphone should be positioned about the length of a biro from the mouth. 
- 3 Avoid wearing 'noisy' clinking jewellery. 

- 4 During the lesson the Campus S will *automatically* adjust to louder voice levels so you need not change the way you normally talk. 
- 5 During the lesson try not to draw attention to the MLxS user but do help them by always repeating questions from the floor.
- 6 Be aware that you have a direct sound link to MLxS users in the room and mute the microphone when you are talking to others.

Tutor courtesy card *continued*

Campus SX/MicroMLxS

Campus SX controls


In normal use, it should not be necessary to use the controls – we explain them here in case you need to use them!




To avoid unintentional channel changing you can lock the control buttons as below.



Alternatively, another solution is to ask your audiology contact to programme the Campus SX with only one channel – to learn how, visit: www.connevans.com/phonak/tx


When not addressing the radio aid system user, always mute the microphone with a short press of the ON/OFF button (don't forget to toggle back to normal afterwards!). When the mic is muted, the channel no. is replaced by a rotating 'O'. 



The battery status is shown in the display and a flat battery is shown as an empty battery icon.  The Campus SX needs daily charging.

Connection facilities into AV equipment are available, please discuss your requirements with your audiology support contact.



After the lesson, if you are not returning the transmitter to the user, remember to turn it off before going to the staff room or toilet! 

The Campus SX has a transmitting range of several classrooms.

Thank you for your time.

PS – general interest suggestion: ask your audiology contact to demonstrate what a hearing aid sounds like and how the Phonak MLxS system helps.