


Silent headphones

The silent headphones provide an inductive output for use with the hearing aid 'T' input.



The silent headphones can be used with any equipment with a suitable headphone socket eg. personal radios, minidisks, CD players, cassettes, talking book machines etc.

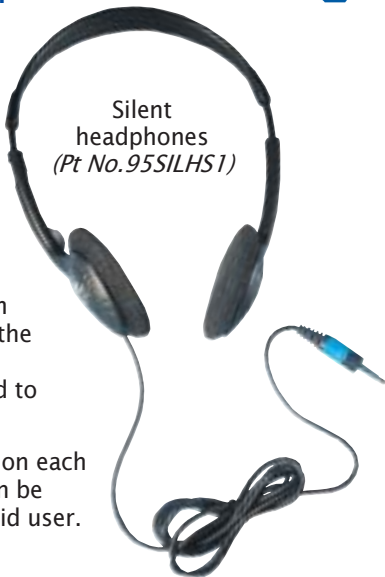

Simply plug the 3.5mm jack plug directly into the headphone socket and switch your hearing aid to the 'T' setting.

Because there is a coil on each side a stereo signal can be heard by the hearing aid user.

The silent headphones are position sensitive - you need to experiment by moving them to get the best signal.

*Can also be used with 3 metre stereo extension lead
(Pt No. ★A121A★)*



CONNEVANS LIMITED

54 Albert Road North, Reigate, Surrey RH2 9YR

Helpline 01737 247571

help@connevens.com