

1 CONNECT up the system as it will be used & **TURN ON**.

Remember to check that connections are pushed home.



2 INVESTIGATE any reported operating problems from yesterday.

Intermittent sound problems can be difficult to trace, ask the user to explain what happened when.



3 TEST the overall system.

Give the transmitter to a friend or place it somewhere a few metres away where it will pick up some identifiable sound and listen to the output of the hearing aid.

If you are the hearing aid user, make sure that you can hear a clear sound from the transmitter. If you are a hearing helper, use a stetoclip and attenuator to listen to the hearing aid.



4 RECEIVER Gently check the connections between the MLxi receiver, the shoe and hearing aid to check that the sound does not break up or crackle.

Worn hearing aid contacts are a common source of problems, if necessary return the hearing aid to the clinic for re-casing.



5 TRANSMITTER While still listening to the hearing aid, ask your helper to gently wiggle the lavalier neck strap on the ZoomLink+ transmitter make sure that the sound does not break up or crackle. If necessary return for repair.



6 BATTERY STATUS Use a battery tester to check the condition of the hearing aid battery. Look at the ZoomLink+ display to see that there is enough power for the time that the system will be used - but of course, you did check & recharge last night!

7 RE-ORDER spares as required - always keep spare shoes and hearing aid batteries etc.

In case of difficulty, consult the ZoomLink+ User Guide which contains much useful help. You will also find lots of information on the Connevans website.



1 The ZoomLink+ has a battery life of approx 10 hours so we advise charging daily.

2 Check that the ZoomLink+ unit on charge has a flashing battery icon. If the battery icon is not flashing then the unit is not charging - check the connections.




3 Do not waste time unnecessarily disconnecting items. You will only have to put them all back together in the morning and it saves general wear and tear.

4 As the MLxi receiver takes its power from the hearing aid battery it is very important that the battery is in good condition. A low hearing aid battery will result in increased hiss and reduced range - the normal life of a hearing aid battery will be more than halved when used with an MLxi.

If in doubt, fit a new battery in the hearing aid.

Handy hint: The ZoomLink+ is supplied with a USB cable which can be used to charge the transmitter from a computer - ideal if you don't have your mains charger handy.

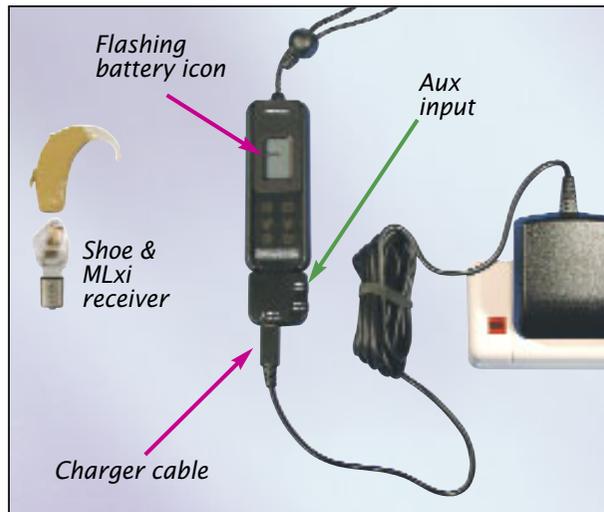

Lavalier aerial
Adjust length by squeezing clip and pulling the free end of the cord.

On off



ZoomLink+ adaptor
Slide in carefully - do not lever in and out or you may cause damage.



BZOPMLICK V 2.0