



Connevens.info

Solutions to improve the quality of life

*Offering you choice
Helping you choose*

This product may be purchased from Connevens Limited
secure online store at www.DeafEquipment.co.uk

DeafEquipment.co.uk



Disabling the Vibrating Pad from the Strobe Function to enable the use of the Strobe only

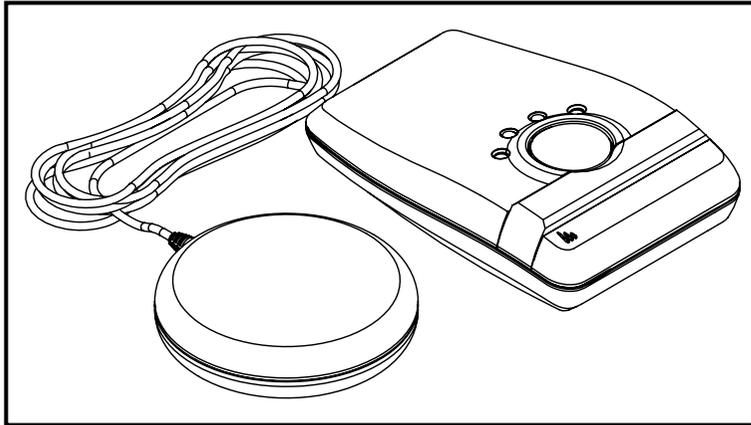


Fig.1

To set up straight out of the box

Plug the strobe in but do not switch it on at the socket. Make sure that the battery is disconnected from the strobe unit. Plug the power cord into the back of the strobe (**Fig.2**). Press and hold the test button. Whilst doing so, switch the power on and continue to hold the test button. After about 5 seconds, the strobe will beep and both red LEDs will flash. You can now release the test button and connect the battery. At this stage you can learn-in the strobe into a network as normal. No fault conditions, indicated by a flashing amber LED, should be present even with the vibrating pad disconnected.

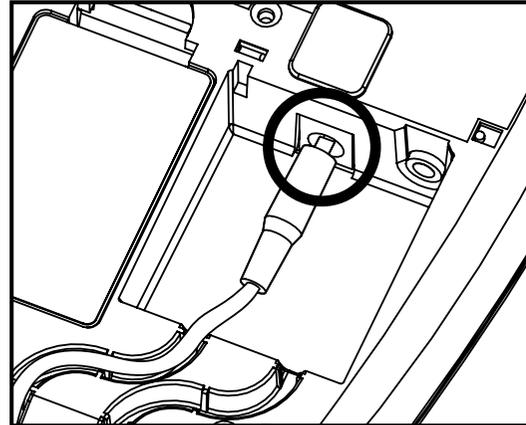


Fig.2

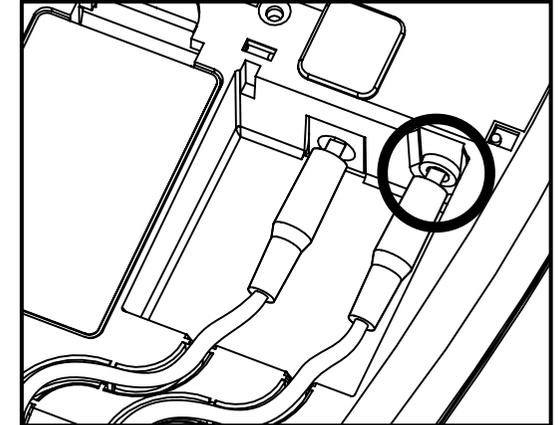


Fig.3

To remove the vibrating pad from an established network

Switch the power off at the socket and disconnect the battery. Disconnect the vibrating pad from the strobe unit (**Fig.3**). Press and hold the test button of the strobe. Whilst doing so, switch the power on and continue to hold the test button. After about 5 seconds, the strobe will beep and both red LEDs will flash. You can now release the test button and connect the battery. No fault conditions, indicated by a flashing amber LED, should be present even with the vibrating pad disconnected.

For further assistance call:
Technical Support
Tel. **0800 141 2561**

PLEASE NOTE: The vibrating pad can be reconnected at any point and will operate correctly. If the vibrating pad is then removed the system will show a fault, indicated by a flashing amber LED once per second. If this occurs and you require the strobe to operate independently, please follow the steps above.